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Face mask Packaging and labelling Requirements

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(On the label) Name, trademark or any other means of identification of the manufacturer or supplier and their address and email address.

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(On the label or Instructions for use) Composition, substances or materials used in its
manufacture

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(On the label) Recommended period of use for continuous use (usually 4 hour) and the expiration date in case the product loses some of its qualities over time.

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(On the label or Instructions for use) Size of the face mask (Adult size) and indicate if it is a disposable face mask.

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(On the label) Place of origin (Made in China)

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(On the label) Reference of the standard: “ UNE 0064-1: 2020. Non-reusable hygienic masks. Requirements for materials, design, preparation, marking and use. Part 1: For adult use ”

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(On the label) “ Warning: This device is not a medical device within the meaning of Directive 93/42 or Regulation EU / 2017/745, nor is it a personal protective equipment (PPE) within the meaning of Regulation EU / 2016/425. This product is intended for a population without symptoms of disease ”

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(In the Instructions for use) How to use

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This face mask is for personal use and disposable

1. Wash your hands with water and soap or a hydroalcoholic gel before manipulate the face mask
2. Take out the face mask and identify the superior part.
3. Put the face mask with the nose clip on top of your nose
4. Place the elastic ear bands behind each ear without crossing them
5. Adjust the face mask until your chin.
6. Use your thumb and index to adjust the nose clip until the mask fits well and it 's comfortable to use
7. Verify that the face mask covers efficiently from the nose to the bottom of the jaw and feels comfortable.
8. Do not touch the face mask once is adjusted. If you need to touch the face mask, wash your hands with water and soap or a hydroalcoholic gel before manipulate it.
9. To remove the mask: wash your hands with soap and water or a hydroalcoholic solution, remove without touching the front of the mask and dispose of it in a container. After disposing the mask wash your hands with soap and water or a hydroalcoholic solution.

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