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digital indication:

8.8 8	alcohol content
“ c ” shows circularly on the lcd	ready for blow
“ ” shows circularly on the lcd	analyze the concentration, please wait
fff	sensor fault

2. operation instruction

2.1. slide down the battery compartment on the back side, insert 3pcs aaa 1.5v batteries inside according to battery polarity indication and close it.

2.2. press the switch button and last for 1s, the tester would be power on with a buzzer brief ringing, ‘ wait ’ shows on down side of screen, meantime it shows 100 to 000 countdown, please wait now, otherwise its accuracy will be not good.

until “ wait ” disappears, tester will send out a ring, meantime shows “ blow ” and “ c ” gliding, now the test could begin.

remarks: its countdown speed will depend on concentration of last test, if concentration is high, it will be slow. if concentration is low, the countdown will be very quick.

2.3. have a deep breath before testing, and blow directly to gas entry when “ c ” circularly shows, stop blow when heard its buzzer ring.

2.4. after blowing finished, the lcd screen will show “ ” circularly, meantime shows “ test ” , this means the tester is analyzing the result, do not blow at this time. after analyze the screen will show the testing results.

it will turn off automatically after keep results for several seconds.

2.5. if a new testing needed, press the switch button again and repeat the above step 4.2 ~ 4.4.

3. notification

3.1. avoid any fall or strong shock.

3.2. if noise gas with high concentration existed, may the tester won ' t work normally.

3.3. if testing is done under low voltage, certain error will be existed between the real value.

3.4. to ensure the testing result, please wait 15 mins to take the testing after your drinking.

3.5. the result may be unreliable for your first test if long time storage, please try second time.

3.6. do not keep the tester in the environment of corrosive gas (chlorine etc) for use or deposit, also in other bad surroundings.

3.7. do not blow smoke to tester directly, especially the smokers.

3.8. when in consecutive testing, if blow immediately after drink, this concentration is very high, there will be some rudimental in mouthpiece. please wait for at least 1 min to take second test, or, blow clean air for 3 seconds to clean out its mouthpiece.

3.9. after long time use, there may be dirt on the tester, please use clean cloth to wipe off. do not use any hard object or the solvent with any corrosive ingredient.

remarks: common fault and solving solutions

fault	possible reason	solution
no display on lcd	incorrect battery installation	insert battery correctly according to polarity
	low battery seriously	new battery replacement
	circuit fault	contact distributor
no response to detection gas	not completewarm-up	waiting completewarm-up
	circuit fault	contact distributor
fff display then power off	sensor fault	contact distributor
low voltage display then power off	low battery seriously	new battery replacement

to keep the continuous improvement of the products, we reserve the right to improve design without prior notice.

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