排毒美容必备排便清肠器 Jian Ke DIY Rectal Syringe

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for a better price, please feel free to contact us.

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1 xpump, 2 x rubber tubes, 1 x container, 1 x nozzle

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this product is used for bowel cleansing and rectal drug administration. jian ke rectal syringe is easy-to-use. the whole process can be diy. all components are fully disinfected according to the national regulation.

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a.) medical use such as, stimulating defecation, rectal drug administration and etc.

b.) personal care such as, removal toxin from the body

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safe. the flushing rate can be controlled

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1.connect the container and pump with one tube.

2.use the other tube to connect the pump and the nozzle.

3.softly place the nozzle into the rectum.

4.press the pump to put the fluid from the container to the rectum.

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wash hands before using the product or wear disinfected glove

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wash the container before and after use.

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personal use only.do not share the same product with others.

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the washing fluid should be instructed by physician.

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can be lubricate the nozzle with olive oil or soap water.

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children age under 12 should be used under physicians supervision.

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two years.

for more details, please feel free to contact us.

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