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5. Promote growth and reduce scarring. Chitin by activating the body ECF and FGF growth factor, promotes

epithelial cell growth. By reducing the formation of collagen and inhibiting local hyperplasia tissue, chitin can effectively reduce the wound scar formation.

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Mainly used for fixing I.V catheter and general protection for post-operative incision wound.

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After cleaning and disinfecting the wound, please take the dressing out of the bag.

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Sterile , disposable.

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Damaged package is prohibited using

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3.No substitute for suture,hemostasis,and skin disinfection.

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4.Not suitable for arterial catheter fixation.

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Prohibit using for infected wound and ulcers.If the wound is infected such as swelling ,effusions ,redness and fever appears,please stop using or increase the replacement frequency according to the doctor diagnosis.

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6.Please select the appropriate size of dressing to paste on the skin naturally.No stretch is allowed in case tension damages to the skin.

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7.When removing the dressing ,please note that neither the catheter nor other devices will not be ripped off.